

## IPERMAR

### INDICATIONS

**Ipermar** nasal wash solution, from the IN-NOSE line, is a medical device consisting of a hypertonic, sterile, preservative-free and ready to use solution, based on sale marino integrale di Cervia, Acqua Madre salina delle Terme di Cervia and hyaluronic acid, packed in a strip of 5 single-dose vials of 5 ml.



**Ipermar** solution, thanks to the combination of sala marino integrale di Cervia and Acqua Madre salina delle Terme di Cervia, is indicated:

- For performing nasal wash in all states of nasal congestion due to nasopharyngeal conditions such as colds, rhinitis, sinusitis, allergic rhinitis, etc.. as it clears the nose of excess mucus, allowing improved breathing in both in adults and children.
- To reduce the appearance of infectious overlays when nasal congestion represents the main pathogenic cause of respiratory inflammations, as it helps mucus removal and mucous membrane cleaning.
- To be used daily to clean the nasal cavities, even in the absence of pathological state, to guarantee the maintenance of mucociliary clearance, as it complies with the physiological balance of the mucous membrane and helps the removal of allergens, smog, germs, and bacteria giving a sensation of immediate comfort.
- To moisturise the respiratory tract thanks to the presence of hyaluronic acid, which can maintain normal tissue hydration. Hyaluronic acid interacts with the surface of the mucosa, holding water and making the wash solution usage more pleasant.

**Ipermar** solution therefore offers relief from the nasal congestion, helps prevent the development of infections thank to its mechanical cleansing action and cleanses the nasale mucous membranes from everyday impurities.

### INSTRUCTION FOR USE:

1. Blow the nose, before performing the washing;
2. Separate the single-dose vial from the strip;
3. Shake gently the single-dose vial before the use;
4. Twist and break the cap of the vial;
5. Recline your head to one side, gently insert the end of the vial into one nostril and close the other nostril with your finger. Instill the solution by squeezing the vial and inhale simultaneously; In case of babies or uncooperative children, it is recommended to do this while lying down;
6. Massage the outer side of the nostril;

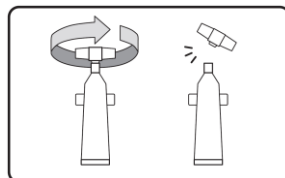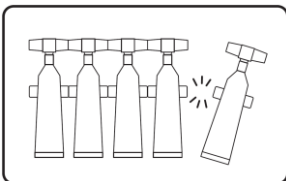

7. Repeat the same operation for the other nostril as well;
8. The use of the vial in the upside-down position eases the spill of the solution;

9. The solution can also be dispensed, following the same method of use, using nebulising accessories consisting of a syringe and nebulizer. In this case, it is recommended to follow the directions of the accessory.

**FREQUENCY OF USE:**

Adults: 1-2 vials per nostril, two or three times a day or according to the doctor prescription.

Children from 2 years old: 1 vial per nostril, two or three times a day or according to the doctor prescription.

**COMPOSITION:**

Sale marino integrale di Cervia 2%, Acqua Madre salina delle Terme di Cervia 1%, Sodium Hyaluronate, Sodium phosphate monobasic monohydrate, Sodium phosphate dibasic dihydrate, purified water.

**PACKAGING:**

The box is composed of 4 strips of 5 single-dose vials. Every single-dose vial is filled at 5 ml.

**WARNINGS:**

- ✓ The product is for nasal use only, do not ingest or use otherwise than the indication of use;
- ✓ Consult your doctor during pregnancy, lactation and in children under two years of age;
- ✓ Clean your hands carefully before using the device;
- ✓ Do not use the product in case of known incompatibility to even one single component of the formulation;
- ✓ In case of adverse reactions during use, immediately suspend the treatment and consult a doctor;
- ✓ The administration of nasal medication concurrently with the nasal wash solution should be carried out under the supervision of a doctor who can evaluate the possible interactions;
- ✓ Keep out of the reach of children;
- ✓ Do not use the product after the expiration date indicated on the package;
- ✓ Once the vial is open immediately drop out the content and do not reuse the remaining solution;
- ✓ Keep unused vials inside the box;
- ✓ In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device;

**VALIDITY:**

36 months with the package intact.



Enable Innovations S.r.l. - Via Claudio Tolomeo n°20/4; 41012 Carpi (MO), Italia



**Optional information:**



XXXXXX