

## IPERMAR AEROSOL




### INDICATIONS

**Ipermar Aerosol** from the IN-NOSE line, is a Medical Device for inhalation use consisting of a hypertonic, sterile, preservative-free and ready to use solution, based on hyaluronic acid, Acqua Madre salina delle Terme di Cervia and Sale marino integrale di Cervia, packed in a strip of 5 single-dose vials.

**Ipermar Aerosol** is indicated in the treatment of acute and chronic alterations of the upper respiratory tract due to nasopharyngeal disorders such as colds, rhinitis, cough, sinusitis, and allergic rhinitis. **Ipermar Aerosol**, thanks to the hypertonia, conferred by the combination of Sale marino integrale di Cervia and Acqua Madre salina delle Terme di Cervia, provides a fluidifying, decongestant and eutrophic activity for the respiratory mucosa, which restores the mucociliary clearance, favouring the removal of excess mucus and recovering the mucosal integrity. The presence of hyaluronic acid restores the level of hydration, giving relief to the irritated mucosa. **Ipermar Aerosol** is optimal also for the cleaning of the nasal cavity and of the upper respiratory tract.

### INSTRUCTION FOR USE:

1. Blow the nose before starting the nebulization;
2. Separate the single-dose vial from the strip;
3. Shake gently the single-dose vial before the use;
4. Twist and break the cap of the vial;
5. Gently press the single-dose vial and put all the solution in the nebulizer ampoule, without diluting it;
6. After the nebulization, do not use any residue in the vial and/or in the nebulizer ampoule.

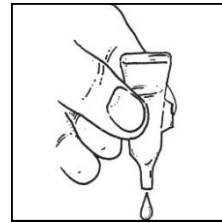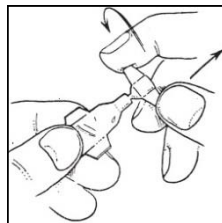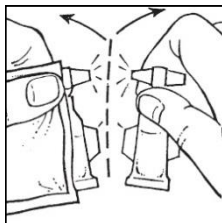

### FREQUENCY OF USE:

Nebulize a 3 mL or 5 mL vial from one to two times a day or according to the doctor's opinion. The product can be used with piston and ultrasonic nebulizer.

### COMPOSITION:

Sale marino integrale di Cervia 2%, Acqua Madre salina delle Terme di Cervia, Sodium Hyaluronate, Sodium phosphate monobasic monohydrate, Sodium phosphate dibasic dihydrate, purified water.

### PACKAGING:

The box is composed of 2 strips of 5 single-dose vials. Every single-dose vial is filled at 3 ml.

The box is composed of 2 strips of 5 single-dose vials. Every single-dose vial is filled at 5 ml.

**WARNINGS:**

- ✓ The product is for inhalation use only, do not ingest or use otherwise than the indication of use;
- ✓ Consult your doctor during pregnancy, lactation and in children under two years of age;
- ✓ Do not use in children under 2 years, because in this age the physiological characteristics of the respiratory tract lead to a limited drainage capacity of the bronchial mucus;
- ✓ Clean your hands carefully before using the device;
- ✓ Do not touch other surfaces with the tip of the vial, during the instillation of the solution in the cup;
- ✓ Do not use the product in case of known incompatibility to even one single component of the formulation;
- ✓ In case of adverse reactions during use, such as bronchospasm, coughing or any irritation, immediately suspend the treatment and consult a doctor;
- ✓ The administration of inhaled medication concurrently with the inhalation solution should be carried out under the supervision of a doctor who can evaluate the possible interactions;
- ✓ Keep out of the reach of children;
- ✓ Do not use the product after the expiration date indicated on the package;
- ✓ Once the vial is open immediately drop out the content and do not reuse the remaining solution;
- ✓ Keep unused vials inside the box;
- ✓ In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

**VALIDITY:**

36 months with the package intact.




Enable Innovations S.r.l. - Via Claudio Tolomeo n°20/4; 41012 Carpi (MO), Italia



0425



STERILE A

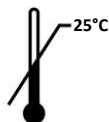**Optional information:**

XXXXXXXX