

**INDICATIONS**





**INNODROPS BB** from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a sterile isotonic saline solution, with physiological pH, preservative-free based on hyaluronic acid, glycerin and distilled waters contained in a strip of 5 single-dose vials filled at 0,5 ml.

The combination of hyaluronic acid and glycerin confers to the eye drops a protective, hydrating and humectant action that helps to stabilize and reinstate the lacrimal film and reduce the friction during blinking.

**INNODROPS BB** is intended to relieve from the ocular redness and fatigue protecting the eye from environmental and atmospheric factors (pollution, smog, sea water, chlorine water, wind, dust, smoke, dry climate, air conditioning) or from visual stress due to prolonged use of electronic devices (computer, smartphone, tablet), intense study, prolonged driving, and prolonged exposure to sunlight. **INNODROPS BB** is a refreshing solution studied for people suffering of occasional dry eye or lack of tearing. It gives prompt relief and restores the daily wellness. **INNODROPS BB** can be instilled before the application and during the use of contact lenses to improve application and comfort.

**INSTRUCTION FOR USE:**

1. Open the protective foil pouch.
2. Carefully clean hands before the use.
3. Separate the single-dose vial from the strip and open it by twisting the cap.
4. Gently press the single-dose vial and instill 2/3 drops per eye.
5. After the instillation, close the vial and reuse the residue solution within 12 hours;
6. Store the unused vials in their original package.
7. The product is single patient to avoid possible contamination.
8. The product can be used with contact lenses.

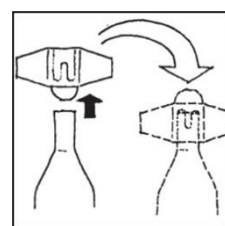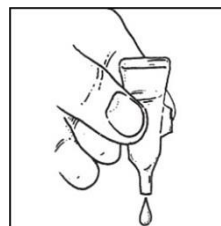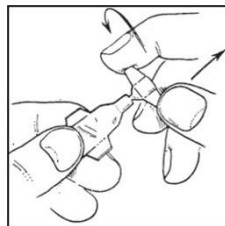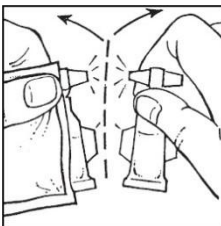

**FREQUENCY OF USE:** Instill 2/3 drops per eye, 2-3 times a day or according to the doctor's opinion.

**COMPOSITION:** Hyaluronic acid 0,15%, glycerin 0,2%, Vaccinium Myrtillus distilled water, Hamamelis Virginiana distilled water, Chamomilla Recutita distilled water, Euphrasia Officinalis distilled water, sodium chloride, sodium phosphate dibasic dodecahydrate, sodium phosphate monobasic monohydrate, purified water.

**PACKAGING:**

The box is composed of 2 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 3 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 5 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 6 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.


### WARNINGS:

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use.
- Once open, the single-dose vial can be reclosed with its cap and the residue solution can be reused within 12 hours to avoid possible contamination of the product.
- Keep out of reach and sight of children.
- During the instillation do not touch the eye or other surfaces with the nozzle of the single-dose vial.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- If necessary, check with your optician for the compatibility with the contact lenses use.
- In case of adverse reaction during the use, immediately suspend the treatment and consult your doctor.
- Consult your doctor for use during pregnancy, lactation and for children.
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In case of compromised ocular surface or damage cornea consult your doctor.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

**VALIDITY:** 36 months with the package intact.



: Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

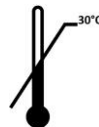

### Optional pictogram



: XXXXXXXX