

## F1a multidose

### INDICATIONS

**INNODROPS F1a** from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a sterile isotonic saline solution, at low viscosity, with physiological pH and preservative-free, based on Perilla Frutescens dry extract and hyaluronic acid, contained in a 10 ml preservative-free bottle.

The Perilla Frutescens dry extract, thanks to the presence of polyphenols, confers an antioxidant action that helps to relieve from the states of redness and irritation caused by oxidative stress, while the hyaluronic acid, which is a highly biocompatible polysaccharide polymer, reinstates the aqueous layer of the lacrimal film thanks to its hydrating action.


**INNODROPS F1a** is intended to alleviate from ocular redness and irritation caused by atmospheric factors (dry climate, wind) or environmental factors (pollution, dust, allergens, smoke) or from visual stress due:

- to prolonged use of electronic devices (computer, smartphone, tablet),
- contact lenses
- intense study or prolonged driving,
- prolonged exposure to sunlight.

**INNODROPS F1a** can be instilled before the application and during the use of contact lenses to improve application and comfort.

### INSTRUCTION FOR USE:

1. Break the safety seal of the container
2. Carefully clean hands before the use
3. Remove the protective cap
4. Gently press the container and instill 2/3 drops per eye
5. After the instillation, close the container, store it in the original package and reuse the residue solution within 3 months
6. The product is single patient to avoid possible contamination
7. The product can be used with contact lenses

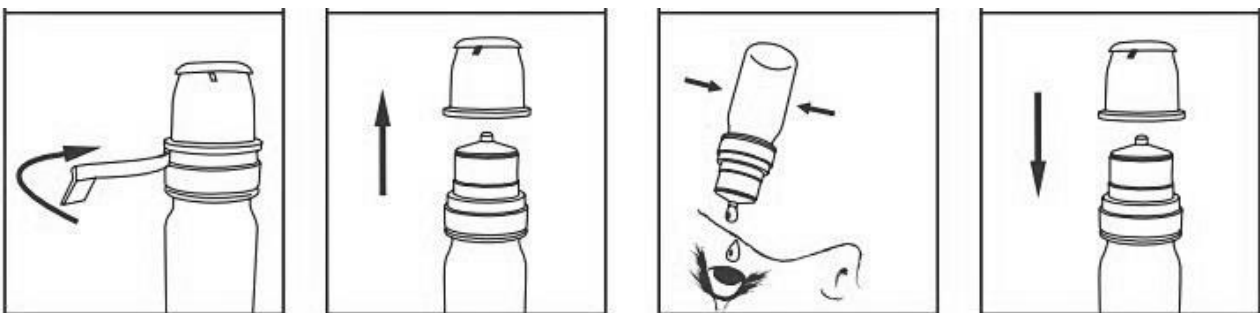

**FREQUENCY OF USE:** Instill 2/3 drops per eye, 2-3 times a day or according to the doctor's opinion.

**COMPOSITION:** Perilla Frutescens dry extract ( $\geq 2,5\%$  in polyphenols) 0,1%, hyaluronic acid 0,1%, sodium chloride, sodium phosphate dibasic anhydrous, sodium phosphate monobasic monohydrate, purified water.

**PACKAGING:** the box is composed of a 10 ml preservative-free bottle. The container is reclosable.

**Subject:****INNODROPS F1a multidose eye drops**

Rev.05 January 2025

Page 2/2

**WARNINGS:**

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use
- Once open, the container can be reclosed with its cap and the residue solution can be reused within 3 months to avoid possible contamination of the product
- Before instillation check the dispenser hole; if present a residue, clean with a wet cloth and expel one droplet before use
- Do not use the product if the packaging box or the container are damaged or not perfectly intact
- Keep out of reach and sight of children
- During the instillation do not touch the eye or other surfaces with the nozzle of the bottle
- Do not use the product in case of known incompatibility to even one single component of the formulation
- If necessary, check with your optician for the compatibility with the contact lenses use
- In case of adverse reaction during the use, immediately suspend the treatment and consult your doctor
- Consult your doctor for use during pregnancy, lactation and for children
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions
- In case of compromised ocular surface or damage cornea consult your doctor
- Do not use the product after the expiration date indicated on the package
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

**VALIDITY:** 36 months with the package intact

: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

**Optional pictogram**

: XXXXXXX