

F1b

INDICATIONS

INNODROPS F1b from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a sterile hypotonic saline solution, at medium viscosity, with physiological pH and preservative-free, based on Perilla Frutescens dry extract and hyaluronic acid, contained in a strip of 5 single-dose vials filled at 0,5 ml.

The Perilla Frutescens dry extract, thanks to the presence of polyphenols, confers an antioxidant action that helps to relieve from the states of redness and irritation caused by oxidative stress, while the hyaluronic acid, which is a highly biocompatible polysaccharide polymer, stabilizes and reinstates the aqueous layer of the lacrimal film thanks to its lubricant and hydrating action.





INNODROPS F1b is intended to:

- Relieve and contrast temporary ocular discomfort as redness, dryness, sensation of external bodies due to atmospheric and environmental conditions (pollution, smog, sea water, chlorine water, wind, dust, smoke, dry climate, air conditioning).
- Relieve ocular fatigue from visual stress such as prolonged use of electronic devices (computer, smartphone, tablet), contact lenses, intense study or prolonged driving or prolonged exposure to sunlight.
- Relieve the sensation of occasional dry eye related to blepharitis and it helps the removal of any secretions.
- Relieve states of allergic conjunctivitis and it promotes the removal of any secretions.

INNODROPS F1b can be instilled before the application and during the use of contact lenses to improve application and comfort.

INSTRUCTION FOR USE:

1. Open the protective foil pouch
2. Carefully clean hands before the use
3. Separate the single-dose vial from the strip and open it by twisting the cap
4. Gently press the single-dose vial and instill 2/3 drops per eye
5. After the instillation, close the vial and reuse the residue solution within 12 hours
6. Store the unused vials in their original package
7. The product is single patient to avoid possible contamination
8. The product can be used with contact lenses

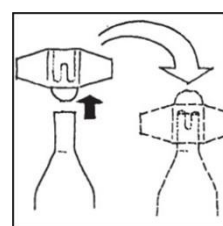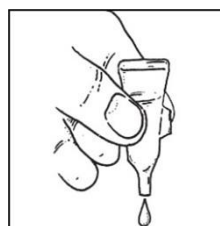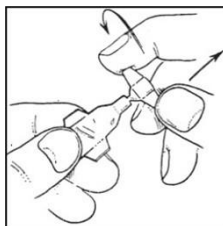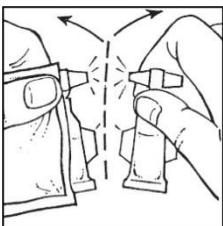


FREQUENCY OF USE: Instill 2/3 drops per eye, 2-3 times a day or according to the doctor's opinion.

COMPOSITION: Perilla Frutescens dry extract ($\geq 2,5\%$ in polyphenols) 0,1%, hyaluronic acid 0,25%, sodium chloride, sodium phosphate dibasic anhydrous, sodium phosphate monobasic monohydrate, purified water.

PACKAGING:

The box is composed of 2 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

	Package Leaflet	
Subject:	INNODROPS F1b eye drops	
Rev.06 January 2025	Page 2/3	

The box is composed of 3 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 5 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 6 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.


WARNINGS:

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use
- Once open, the single-dose vial can be reclosed with its cap and the residue solution can be reused within 12 hours to avoid possible contamination of the product
- Do not use the product if the foil pouch or the single-dose vials are damaged or not perfectly intact
- Keep out of reach and sight of children
- During the instillation do not touch the eye or other surfaces with the nozzle of the single-dose vial
- Do not use the product in case of known incompatibility to even one single component of the formulation
- If necessary, check with your optician for the compatibility with the contact lenses use
- In case of adverse reaction during the use, immediately suspend the treatment and consult your doctor
- Consult your doctor for use during pregnancy, lactation and for children
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions
- In case of compromised ocular surface or damage cornea consult your doctor
- Do not use the product after the expiration date indicated on the package
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device

VALIDITY: 36 months with the package intact



: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

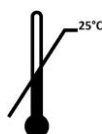

Subject:

INNODROPS F1b eye drops

Rev.06 January 2025

Page 3/3

Optional pictogram



: XXXXXXX

Rev.06 – January 2025