

F2

INDICATIONS

INNODROPS F2 from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a sterile hypotonic saline solution, at high viscosity, with physiological pH and preservative-free, based on Perilla Frutescens dry extract, hyaluronic acid, hydroxypropylmethylcellulose (HPMC) and glycerin contained in a strip of 5 single-dose vials filled at 0,5 ml.

The Perilla Frutescens dry extract, thanks to the presence of polyphenols, confers an antioxidant action that helps to relieve from the states of redness and irritation caused by oxidative stress, while the combination of hyaluronic acid, hydroxypropylmethylcellulose (HPMC) and glycerin, gives to the eye drops an intense lubricant and humectant action for the ocular surface.

INNODROPS F2 is intended for the mitigation of moderate dry eye symptomatology related to:

- Unfavorable atmospheric and environmental conditions such as pollution, smog, sea water, chlorine water, wind, dust, smoke, dry climate, air conditioning.
- Ophthalmic stress due to a prolonged use of eye drops containing preservatives in a chronic local therapy.
- Prolonged use of contact lenses.
- Relieve the sensation of dry eye related to blepharitis and allergic conjunctivitis.
- Insufficient lacrimation
- Prolonged use of electronic devices (computer, smartphone, tablet).

INNODROPS F2 can be instilled before the application and during the use of contact lenses to improve application and comfort.

INSTRUCTION FOR USE:

1. Open the protective foil pouch.
2. Carefully clean hands before the use.
3. Separate the single-dose vial from the strip and open it by twisting the cap.
4. Gently press the single-dose vial and instill 2/3 drops per eye.
5. After the instillation, close the vial and reuse the residue solution within 12 hours.
6. Store the unused vials in their original package.
7. The product is single patient to avoid possible contamination
8. The product can be used with contact lenses.

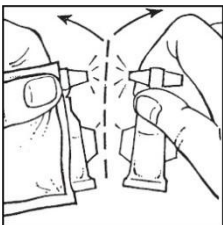


Fig.1: Separate single-dose vial

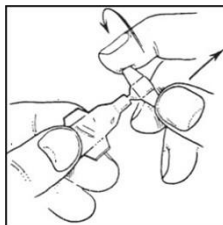


Fig.2: open it by twisting the cap

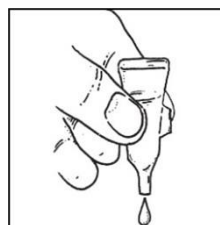


Fig.3: instill the solution

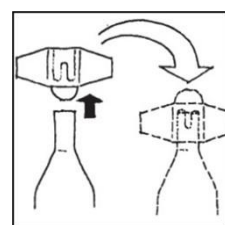


Fig.4: reclose the vials

FREQUENCY OF USE: Instill 2/3 drops per eye, 2-3 times a day or according to the doctor's opinion

COMPOSITION: Perilla Frutescens dry extract ($\geq 2,5\%$ in polyphenols) 0,1%, hyaluronic acid 0,25%, hydroxypropylmethylcellulose (HPMC) 0,3%, glycerin 0,2%, sodium chloride, sodium phosphate dibasic anhydrous, sodium phosphate monobasic monohydrate, purified water.

PACKAGING:

The box is composed of 2 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 3 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 5 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 6 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

WARNINGS:

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use.
- Once open, the single-dose vial can be reclosed with its cap and the residue solution can be reused within 12 hours to avoid possible contamination of the product
- Do not use the product if the foil pouch or the single-dose vials are damaged or not perfectly intact.
- Keep out of reach and sight of children.
- During the instillation do not touch the eye or other surfaces with the nozzle of the single-dose vial.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- If necessary, check with your optician for the compatibility with the contact lenses use.
- In case of adverse reaction during the use, immediately suspend the treatment and consult your doctor.
- Consult your doctor for use during pregnancy, lactation and for children.
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In case of compromised ocular surface or damage cornea consult your doctor.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

VALIDITY: 24 months with the package intact




: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

Subject:

INNODROPS F2 eye drops

Revision 07 - January 2025

Page 3/4

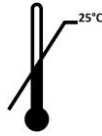

Optional pictogram



: XXXXXXXX