

Subject:**INNODROPS LIPO Eye Drops based on liposomes and medium molecular weight Sodium Hyaluronate, sterile and preservative-free solution in 0,5mL monodose vials**

Rev.03 January 2025

No. Annexes: 0

Pages 1/3

INNODROPS LIPO Eye Drops

INDICATIONS





INNODROPS LIPO from the INNODROPS line is a Medical Device for ophthalmic application consisting of a sterile, slightly hypotonic saline solution, physiological pH, preservative-free, with combination of medium molecular weight sodium hyaluronate and liposomes, packed in 0,5 mL single-vials.

Liposomes in combination with hyaluronic acid provide a functional support to the ocular surface, reconstituting the lipid layer, used for limiting and controlling the tear film evaporation. **INNODROPS LIPO** is also able to regulate and improve the lubrication and hydration of the ocular mucosa.

INNODROPS LIPO is recommended to relieve from ocular redness, dryness and tiredness caused by dry eye syndrome in particular due to the inadequate quality of the lipid layer of the ocular surface, by environmental stress (dust, allergens, smoke, smog, air conditioned) and by ocular stress due to prolonged use of the computer, television and video display terminal. The product can be use even by contact lenses wearers, without having to remove them, to improve eye lubrication and comfort.


INSTRUCTIONS FOR USE:

1. Open the protective foil pouch
2. Carefully clean hands before the use
3. Separate the single-dose vial from the strip and open it by twisting the cap
4. Gently press the single-dose vial and instill 2/3 drops per eye
5. After the instillation, close the vial and reuse the residue solution within 12 hours
6. Store the unused vials in the original package
7. The product is single patient to avoid possible contamination
8. The product can be used with contact lenses

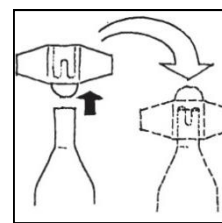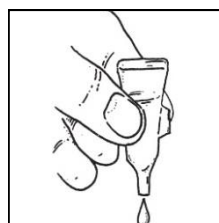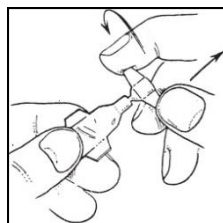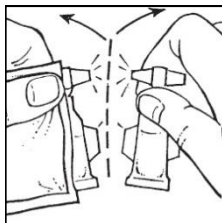

FREQUENCY OF USE: Instill 2/3 drops per eye, 1-2 times a day or according to the doctor's opinion

COMPOSITION: Liposome 0041 1%, Sodium Hyaluronate, Sodium Chloride, Sodium Citrate tribasic dihydrate, Citric Acid monohydrate, Purified water.

	PACKAGE LEAFLET	
Subject:	INNODROPS LIPO Eye Drops based on liposomes and medium molecular weight Sodium Hyaluronate, sterile and preservative-free solution in 0,5mL monodose vials	
Rev.03 January 2025	No. Annexes: 0	Pages 2/3

PACKAGING:

The box is composed of 2 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 3 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 5 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 6 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

WARNINGS:

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use
- Once open, the single-dose vial can be reclosed with its cap and the residue solution can be reused within 12 hours to avoid possible contaminations of the product
- Keep out of reach of children
- During the instillation do not touch the eye or other surfaces with the nozzle of the single-dose vial
- Do not use the product in case of known incompatibility to even one single component of the formulation
- If necessary, check with your optician for the compatibility with the contact lenses use
- In case of adverse reaction during the use, immediately suspend the treatment and consult your doctor
- Consult your doctor for use during pregnancy, lactation and for children
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In case of compromised ocular surface or damage cornea consult your doctor
- Do not use the product after the expiration date indicated on the package
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.
- The product contains soy

VALIDITY: 36 months with the package intact

Subject:

INNODROPS LIPO Eye Drops based on liposomes and medium molecular weight Sodium Hyaluronate, sterile and preservative-free solution in 0,5mL monodose vials



Rev.03 January 2025

No. Annexes: 0

Pages 3/3



: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

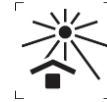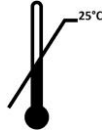

Optional information:



: XXXXXXX