




INDICATIONS

IN-NOSE BLUE SPRAY from the IN-NOSE line is a medical device for nasal application consisting of a sterile hypertonic solution, preservative free, with Tween 80, hyaluronic acid and EDTA, in a spray bottle of 20ml.

Thanks to the combined action of EDTA and Tween 80 in hypertonic solution, **IN-NOSE BLUE SPRAY** is indicated for the cleaning of nasal cavities, in particular for the dilution and mechanical removal of the bacterial biofilm layer typical of chronic rhinosinusitis and cooling syndromes, restoring the mucociliary clearance. The presence of hyaluronic acid, performs a moisturizing and emollient action, reducing the dryness sensation of the nasal mucosa and protecting it from external agents such as smoke, smog, dust, pollen, possible causes of irritative disturbances.

INSTRUCTION FOR USE:

1. Carefully clean hands
2. Blow the nose before the use
3. Shake gently the spray bottle before the use
4. Remove the protective cap
5. Hold the spray bottle with your thumb under the base and the nozzle between two fingers
6. At first use, press the pump a few times, until a uniform spray is emitted
7. Tilt your head back slightly and insert the nozzle into a nostril
8. Spray and breathe gently with your nose at the same time
9. After use, clean and close the nozzle with its protective cap
10. The product is for single patient to avoid possible contamination

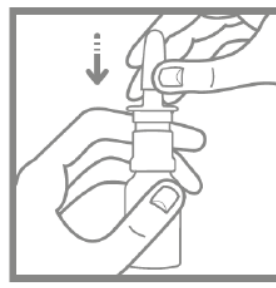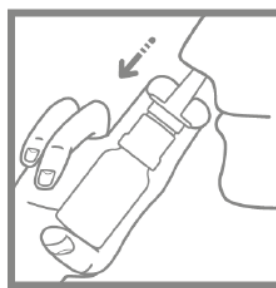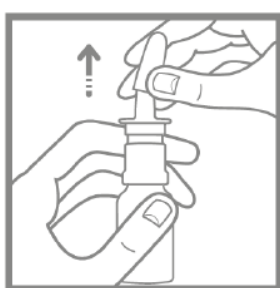

FREQUENCY OF USE: 1-2 nebulization per nostril 2/3 times a day, or according to the doctor's opinion.

COMPOSITION: Sodium hyaluronate, sodium chloride (2,5%), tween 80, EDTA, sodium phosphate dibasic dihydrate, sodium phosphate monobasic monohydrate, purified water.

PACKAGING: the box is composed of a 20 ml spray bottle, reclosable.

WARNINGS:

- The product is for inhalation use only, don't ingest or use otherwise than the indication of use.
- Once the bottle has been opened and used, the residual solution must be used within 3 months to avoid possible contamination of the product.
- Keep out of reach of children.
- Do not touch other surfaces, apart the nasal mucosa, with the dispensing end of the bottle.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- In case of adverse reactions (burning, irritation, etc.) during use, discontinue treatment and consult a doctor or pharmacist.

Subject:**IN-NOSE BLUE SPRAY**

Rev. 02 - January 2025

Pages 2/2

- Consult your doctor for pediatric use and for use during pregnancy and lactation.
- The administration of drugs or medical devices by inhalation concomitantly with the nasal spray should be carried out under the supervision of a doctor who can assess the possible interactions.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

VALIDITY: 36 months with the package intact.

Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italia



25°C

**Optional information**

: XXXXXXX

Rev.02 - January 2025