

INDICATIONS

Specially designed moisturizing solution preservative-free with optimal protection for the eyes.

In clinical trials of practicable suitability, it was found to be most suitable for all conventional gas-permeable, flexible hard lenses and all conventional hydrophilic soft lenses, especially when replaced regularly (every 1-3 months).

For whom is suitable?

The solution is recommended as preventive measure for all persons and especially those with dry or slightly irritated eyes, that may be stressed by environmental factors such as air conditioning, wind, low humidity, air pollution, prolonged working at a computer screen or wearing contact lenses for long periods.

Instructions for use

Instill 1-2 drops per eye into the outer angle of the eyelids as needed. It may be used several times daily, even while wearing contact lenses.

WARNINGS:

- The solution must not be used if you are allergic to any of the ingredients. If the conjunctiva becomes red or other eye symptoms arise, you should immediately consult an eye doctor or contact lens specialist or the pharmacist.
- After opening, to avoid possible contamination of the product, the single-dose vial can be reclosed with its cap and the residue solution can be used within 12 hours; even the bottle can be reclosed with its cap and the residue solution can be used within 3 months.
- To prevent possible microbial contamination of the solution, do not touch the dropper tip of the bottle and the vial.
- If you cannot see as clearly as normal for a short while after using the solution, you should not drive or operate machines until your sight has returned to normal.
- Keep out of the reach of children.
- Consult your doctor for use during pregnancy, lactation and for children.
- The eventual treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In the event of any serious claims or incidents concerning the device, it is necessary to report the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

COMPOSITION: Calcium chloride 1,8 mmol/l, Magnesium sulphate 0,81 mmol/l, Potassium chloride 5,37 mmol/l, Hyaluronic acid 0,075%, hydroxypropylmethylcellulose (HPMC) 0,05%, Sodium chloride, Boric acid, Sodium tetraborate decahydrate, purified water.

PACKAGING:

DROP & SEE MD: The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable. **REF 050368**

DROP & SEE: The box is composed of a 10 ml preservative-free bottle. The container is reclosable. **REF 050369**

The «**DROP&SEE**» and «**DROP&SEE MD**» solutions may also be used temporarily and in exceptional situations as a storage solution for all current types of contact lenses.

VALIDITY: 36 months with the package intact.



: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italia



Rev.02 - February 2025

Optional information