

## Ophthalmic spray SO1

### INDICATIONS

**Ophthalmic spray SO1** is a Medical Device consisting of a sterile, slightly hypotonic saline solution, with physiological pH and preservative-free, based on Liposomes and Acqua Madre Salina, in a 10mL spray bottle.

Liposomes are optimal to restore and stabilize the lipid component present on the eyelid rim, while the essential electrolytes, present in the Acqua Madre Salina, rebalance all the conditions of eye discomfort and tear instability. Thanks to its components, **Ophthalmic spray SO1** is able to regulate, moisturize and refresh the eye mucosa and give functional support to the eye surface. Thus, the lipid layer used to contain and control the evaporation of the tear film is re-established.


The use of **Ophthalmic spray SO1** quickly provides relief to the most common eye discomforts such as burning, foreign body sensation, dryness, redness and fatigue, which can be caused by:

- cases of dry eye syndrome in particular caused by the qualitative inadequacy of the lipid layer of the eye surface,
- unfavorable weather and environmental conditions (air conditioning, dry or very hot climate, heating systems, wind, pollution, dust, smoke, allergens),
- visual stress due to a long period of driving or study, prolonged use of contact lenses or electronic devices (computer, television, display screen equipment, smartphone).

**LIPOFRESH** ophthalmic spray can also be administered during the use of contact lenses to improve eye lubrication and comfort.

### INSTRUCTIONS FOR USE:

1. Carefully clean hands before the use of the spray bottle
2. Gently shake the spray bottle before the use
3. Perform two empty dispenses at first
4. During dispensing, keep the spray bottle in vertical position
5. Gently press on the spray pump and deliver 1-2 sprays, at 10 cm distance, on the closed eyelids.
6. After the instillation, store the spray bottle in the original package and reuse the residue solution within 3 months
7. The product is single patient to avoid possible contamination
8. The product can be used with contact lenses

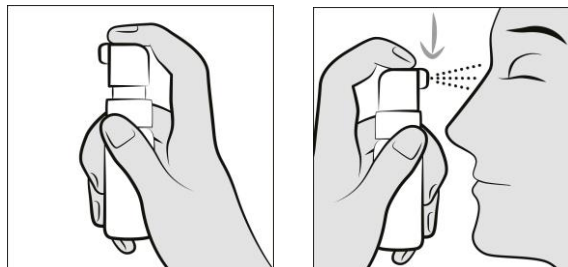

**FREQUENCY OF USE:** We recommend use for adults and children from 3 years. Deliver 1-2 sprays, at 10 cm distance, on the closed eyelids, 3 or 4 times a day, or according to the doctor's opinion.

**COMPOSITION:** Liposomes 1%, Acqua Madre Salina 1%, Sodium chloride, Sodium citrate tribasic dihydrate, Citric acid monohydrate, Purified water.

**Subject:****Ophthalmic spray SO1**

Revision 03 – February 2025

Pages 2/3

**PACKAGING:** The box contains 10mL preservative-free spray bottle**WARNINGS:**

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use
- Once open, store the spray bottle in the original package and the residue solution can be reused within 3 months to avoid possible contamination of the product
- If necessary, consult your doctor for use during pregnancy, lactation and for children under 3 years of age.
- Keep out of reach of children
- Do not spray the product on the open eye
- Before instillation check the dispenser hole; if present a residue, clean with a wet cloth and spray one time before use.
- During the instillation do not touch the eye or other surfaces with the nozzle of the bottle
- Do not use the product in case of known incompatibility to even one single component of the formulation
- If necessary, check with your optician for the compatibility with the contact lenses use
- In case of adverse reaction during the use, immediately suspend the treatment and consult your doctor
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions
- In case of compromised ocular surface or damage cornea consult your doctor
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.
- The product contains soy

**VALIDITY:** 36 months with the package intact

: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

**Optional pictogram**

: XXXXXXX