

## INDICATIONS

**FourHyal Plus** from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a sterile, viscous, slightly hypotonic saline solution, physiological pH, preservative-free, phosphate free, based on hyaluronic acid and Hamamelis distilled water.

The high concentration of Hyaluronic Acid gives the eye increased hydration and, in case of tear film alteration, restores its balance.

**FourHyal Plus** is indicated to protect and moisturize the ocular surface in case of redness, burning and eye fatigue due to:

- dry eye syndrome,
- environmental stress (dust, allergens, smoke, air conditioning, environmental pollution),
- visual stress due to prolonged use of electronic devices (computers, television, video terminals), intense study and prolonged driving.

**FourHyal Plus** is a refreshing solution studied for people suffering of occasional dry eye or lack of tearing. It gives prompt relief and restores the daily wellness. **FourHyal Plus** can be instilled before the application and during the use of contact lenses to improve application and comfort.

## INSTRUCTIONS FOR USE

1. Break the safety seal of the container
2. Carefully clean hands before the use
3. Remove the protective cap
4. Gently press the container and instill 1/2 drops per eye
5. After the instillation, close the container, store it in the original package and reuse the residue solution within 3 months
6. The product is single patient to avoid possible contamination
7. The product can be used with contact lenses



## FREQUENCY OF USE

Instill 1-2 drops per eye, 2-3 times a day or according to the doctor's opinion.

## COMPOSITION

Hyaluronic acid 0,4%, Hamamelis Virginiana distilled water, sodium chloride, sodium citrate tribasic dihydrate, citric acid monohydrate, purified water.

## PACKAGING

The box is composed of a 10 ml preservative-free bottle. The container is reclosable.

## WARNINGS

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use.

**Subject:****FourHyal Plus multidose eye drop**

Revision 02 of January 2025

Pages 2/2

- Once open, the container can be reclosed with its cap and the residue solution can be reused within 3 months to avoid possible contamination of the product.
- Before instillation check the dispenser hole; if present a residue, clean with a wet cloth and expel one droplet before use.
- Keep out of reach and sight of children.
- During the instillation do not touch the eye or other surfaces with the nozzle of the bottle.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- If necessary, check with your optician for the compatibility with the contact lenses use.
- In case of adverse reaction during the use, suspend the treatment and consult your doctor or pharmacist.
- Consult your doctor for use during pregnancy, lactation and for children.
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In case of compromised ocular surface or damage cornea consult your doctor.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

**VALIDITY:** 36 months with the package intact.

: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

**Rev.02 January 2025****Optional information:**

: XXXXXXX