

INDICATIONS

FourHyal from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a sterile, viscous, slightly hypotonic saline solution, physiological pH, preservative-free, phosphate free, based on hyaluronic acid.

The high concentration of Hyaluronic Acid gives the eye increased hydration and, in case of tear film alteration, restores its balance.





FourHyal is indicated to protect and moisturize the ocular surface in case of redness, burning and eye fatigue due to:

- dry eye syndrome,
- environmental stress (dust, allergens, smoke, air conditioning, environmental pollution),
- visual stress due to prolonged use of electronic devices (computers, television, video terminals), intense study and prolonged driving.

FourHyal can be administered in postoperative treatment to relieve symptoms of dry eye and during the use of contact lenses to improve application and comfort.

INSTRUCTION FOR USE

1. Open the protective foil pouch
2. Carefully clean hands before the use
3. Separate the single-dose vial from the strip and open it by twisting the cap
4. Gently press the single-dose vial and instill 1/2 drops per eye
5. After the instillation, close the vial and reuse the residue solution within 12 hours
6. Store the unused vials in their original package
7. The product is single patient to avoid possible contamination
8. The product can be used with contact lenses

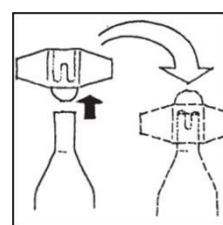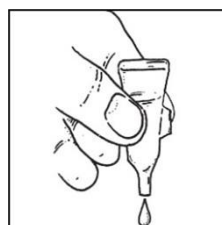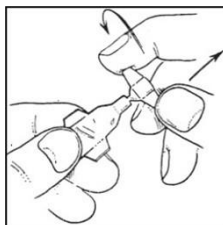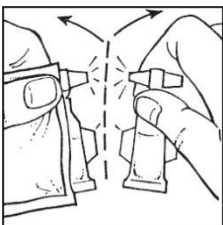

FREQUENCY OF USE

Instill 1-2 drops per eye, 2-3 times a day or according to the doctor's opinion.

COMPOSITION

Hyaluronic acid 0,4%, sodium chloride, sodium citrate tribasic dihydrate, citric acid monohydrate, purified water.

PACKAGING

The box is composed of 2 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 3 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 5 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 6 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

WARNINGS


- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use.
- Once open, the single-dose vial can be reclosed with its cap and the residue solution can be reused within 12 hours to avoid possible contamination of the product.
- Keep out of reach and sight of children.
- During the instillation do not touch the eye or other surfaces with the nozzle of the single-dose vial.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- If necessary, check with your optician for the compatibility with the contact lenses use.
- In case of adverse reaction during the use, suspend the treatment and consult your doctor or pharmacist.
- Consult your doctor for use during pregnancy, lactation and for children.
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In case of compromised ocular surface or damage cornea consult your doctor.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

VALIDITY

36 months with the package intact.



Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

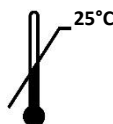

Subject:**FourHyal single-dose eye drops**

Revision 02 of January 2025

Page 3/3

Optional information:

: XXXXXXX