

INDICATIONS

Protection Drop from the INNODROPS line, is a Medical Device for ophthalmic application consisting of a slightly hypotonic sterile solution, with physiological pH and preservative-free, based on Ectoin and Hyaluronic Acid.

Ectoin is a natural molecule able to protect ocular tissues from external damage, including that generated by UV radiation from the sun or artificial blue light emitted by video screens. Thanks to its antioxidant action against free radicals, it inhibits photo-oxidative damage against proteins and lipids, reduces the release of stress mediators and promotes the production of protective agents. It also stabilizes the lipid layer of the tear film, responsible to control tear evaporation.

Hyaluronic acid, a highly biocompatible polymer, replenishes the aqueous layer of the tear film thanks to its moisturizing action. In combination hyaluronic acid and ectoin stabilize the ocular surface by maintaining high water concentrations and preventing dry eye syndrome.





Protection Drop is indicated to protect, stabilize and hydrate the ocular surface in all cases of:

- photo-exposure of the eyes to sunlight, at seaside, at mountains or even daily outdoors
- prolonged use of artificial light sources (PC, tablet, smartphone, TV)
- ophthalmic stress caused by extreme environmental conditions, such as high salinity, temperature changes, dry environment

Protection Drop can be instilled before the application and during the use of contact lenses to improve application and comfort.

INSTRUCTION FOR USE:

1. Open the protective foil pouch.
2. Carefully clean hands before the use.
3. Separate the single-dose vial from the strip and open it by twisting the cap.
4. Gently press the single-dose vial and instill 1/2 drops per eye.
5. After the instillation, close the vial and reuse the residue solution within 12 hours;
6. Store the unused vials in their original package.
7. The product is single patient to avoid possible contamination.
8. The product can be used with contact lenses.

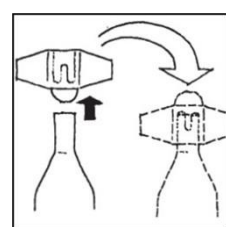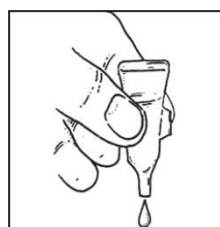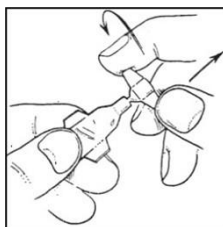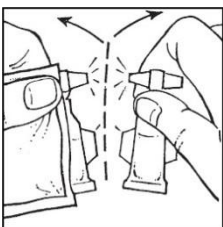

FREQUENCY OF USE: Instill 1-2 drops per eye, 1-2 times a day or according to the doctor's opinion.

COMPOSITION: Ectoin 2%, Hyaluronic acid 0,1%, sodium chloride, sodium phosphate monobasic monohydrate, sodium phosphate dibasic anhydrous, purified water.

PACKAGING:

The box is composed of 2 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 3 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 4 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 5 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

The box is composed of 6 foil pouches each one containing one strip of 5 single-dose vials. Every single-dose vial is filled at 0,5 ml and it is reclosable.

WARNINGS:

- The product is for ophthalmic use only, do not ingest or use otherwise than the indication of use.
- Once open, the single-dose vial can be reclosed with its cap and the residue solution can be reused within 12 hours to avoid possible contamination of the product.
- Keep out of reach and sight of children.
- During the instillation do not touch the eye or other surfaces with the nozzle of the single-dose vial.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- If necessary, check with your optician for the compatibility with the contact lenses use.
- In case of adverse reaction during the use, suspend the treatment and consult your doctor or pharmacist.
- Consult your doctor for use during pregnancy, lactation and for children.
- The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.
- In case of compromised ocular surface or damage cornea consult your doctor.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

VALIDITY: 36 months with the package intact.



: Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy



Rev.02 January 2025

Optional information:



: XXXXXXX