

CHARACTERISTICS

Osmotears Low 280 eye drop is an isoosmolar (slightly reduced salt concentration) sterile solution with high biocompatibility, at low viscosity and physiological pH. The eye drops contain hyaluronic acid and all the essential electrolytes of the tearfilm (nutrition for the cells on the surface of the eye).

Osmotears Low 280 eye drops are packed in a box containing a 10 mL preservative-free reclosable bottle.

INDICATIONS



Osmotears Low 280 eye drop is intended to supplement the eye's natural tear film in cases of reduced tearing, relieving redness, tiredness caused by atmospheric and environmental agents such as dust, allergens etc., as well as ocular stress due to prolonged use of the computer, excessive exposure to light and/or longtime wearing of contact lenses. Thanks to the moisturizing characteristics of hyaluronic acid, and to the presence of essential electrolytes such as magnesium, sodium, potassium, calcium and bicarbonate, which are similar to the tear film, is able to rebalance all those conditions of ocular discomfort and tear film instability.

It is also indicated to cleanse and rinse the eyes, removing any allergens or foreign bodies.

Osmotears Ultra Low 180 eye drop can be administered with contact lenses worn. Instilling a drop before the contact lenses facilitates their application; moreover, the instillation after their application makes them more comfortable, contrasting the feeling of dryness.

INSTRUCTIONS FOR USE

1. Break the safety seal of the bottle
2. Carefully clean hands before the use
3. Remove the protective cap
4. Gently press the body of the container and instil 2/3 drops in each eye
5. After instillation, immediately close the container, store it in the original package and reuse the residue solution within 3 months
6. Store the container in their original package
7. The product is for single patient to avoid possible contamination
8. The product can be used with contact lenses

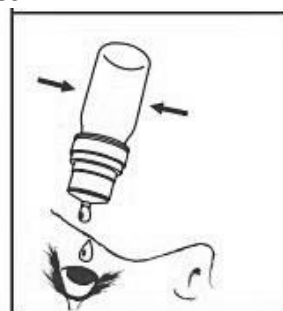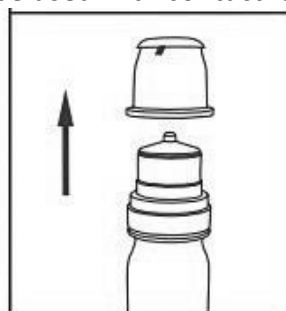

FREQUENCY OF USE: Instill 2/3 drops per eye, 4-6 times a day or as needed

COMPOSITION: Magnesium Chloride, Calcium Chloride, Potassium Chloride, Sodium Bicarbonate, Hyaluronic Acid sodium salt 0.18%, Sodium Chloride, Sodium Phosphate dibasic anhydrous, Sodium Phosphate monobasic monohydrate, purified water.

WARNINGS:

The product is for ophthalmic use only.

Once open, the container can be reclosed with its cap and the residue solution can be reused within 3 months to avoid possible contamination of the product.

In case of need, consult your doctor for pediatric use and for use during pregnancy and lactation.

Before instillation check that there is no residue on the dispenser hole; if present, clean with a wet cloth and expel one droplet before use.

In case of compromised ocular surface or damage cornea consult your doctor.

Keep out of reach of children.

Do not touch the eye or other surfaces with the dispensing end of the bottle during installation.

Do not use the product in case of known incompatibility to even one single component of the formulation.

In case of disturbances during use, immediately suspend the treatment and contact a doctor.

The treatment with ophthalmic drugs or other ophthalmic medical devices must be carried out under the supervision of a doctor who can evaluate possible interactions.


Do not use the product after the expiration date indicated on the package.

In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

VALIDITY: 36 months with the package intact.



: Enable Innovations S.r.l. Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

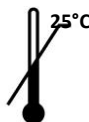

Optional information

