

NASAL NEB

Medical device for nebulizing topical solutions through the nose


INDICATIONS

NASAL NEB is a needle-free medical device designed to allow topical solutions to be administered by nebulization through the nose. It is indicated for the treatment of the nasal cavities, paranasal sinuses and upper airways. **NASAL NEB** is used in the domestic setting for the administration of nasal washes. The conical shape of the nozzle, which is made of soft material, allows it to grip the nostril, ergonomically adapting to both adults and children.

INSTRUCTION FOR USE:

1. Disconnect the device for nasal nebulization NASAL NEB by removing it from the supplied syringe;
2. Open the single-dose vial by twisting the tabs;
3. Connect the syringe to the vial and aspirate the solution contained in the vial;
4. Fix the device for nasal nebulization NASAL NEB by twisting it on the syringe;
5. In children over 2 years of age and adults, insert the tip of the NASAL NEB nasal nebulizer into one nostril and, keeping the chest upright, push the piston of the syringe firmly in to nebulize the solution.
For infants and children under 2 years of age it is necessary to do this lying on their side, keeping the head reclined to one side and inserting the nebulizer gently into the upper nostril.

Repeat steps for the other nostril.

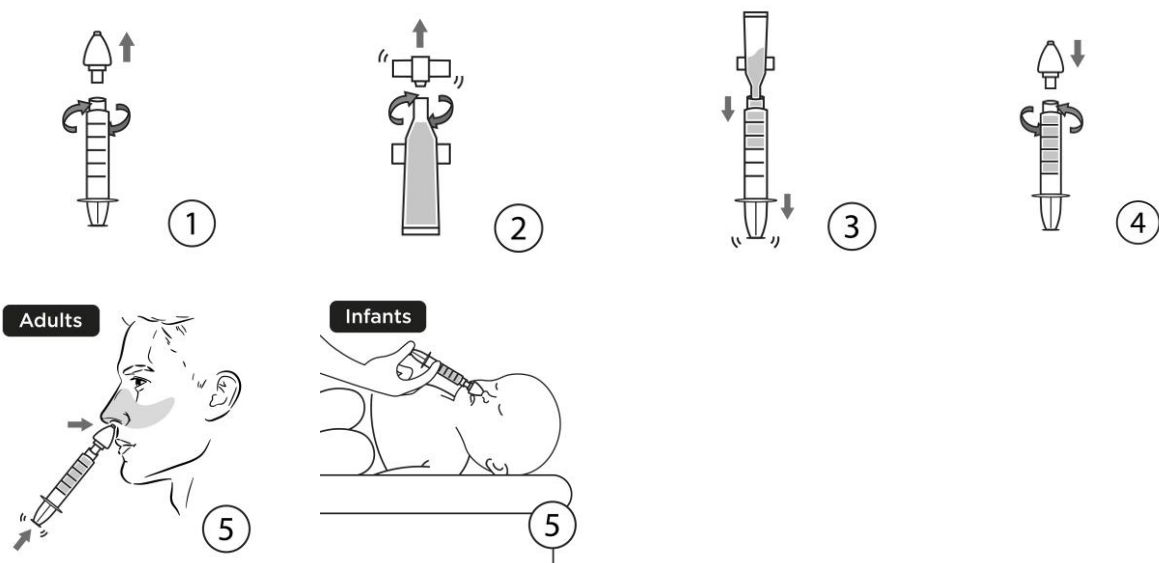

CLEANING OF NASAL NEB

NASAL NEB is a device intended for single-patient use to avoid possible contamination, but it can be used several times. For this reason, it is extremely important that the device is carefully cleaned after each use to ensure durability and proper functioning. After use, fill the syringe with running water and nebulize completely. Repeat this operation a couple of times. Finally, dry the nasal nebulizer thoroughly with a soft cloth or warm air jet. The device should be stored in a dry place, away from light and heat sources.

Subject:**Medical Device for Nasal Nebulization_NASAL NEB + Syringe**

Revision 00 January 2025

Page 2/2

WARNINGS


- The product is for nasal use only, do not use otherwise than the indication of use;
- Use the nebulizer within one month after opening the package and make a maximum of 100 nebulizations;
- Do not use the device under 3 months of age;
- For reuse, clean according to the instructions in the section "Cleaning of NASAL NEB";
- For people with disabilities, the elderly and children, the assistance of an adult person is recommended during use;
- Keep out of reach of children;
- The device is single use patient to avoid possible contamination;
- Do not use if a nosebleed is in progress or has just ended;
- Do not use the device in case of known hypersensitivity to any component;
- Do not force the device into the nasal cavity;
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

PACKAGE CONTENTS

- 1 device for nasal nebulization NASAL NEB
- 1 Luer Lock Syringe 5mL without needle



Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy

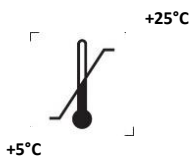**Optional information**

: XXXXXXXX