

Hypertonic solution for nebulisation based on Hyaluronic Acid, plant extract of Perilla and distilled vegetable waters of Boswellia, Grindelia and Elderberry.

10 single dose 3 mL containers for aerosol.

FEATURES

Inhalation Solution S2 is a Medical Device for inhalation containing plant extract of Perilla, distilled vegetable waters of Boswellia, Grindelia and Elderberry in combination with Hyaluronic Acid and Sodium Chloride 2.0%.

Inhalation Solution S2 is a ready for use solution for aerosols, mildly hypertonic and sterile.




INDICATIONS

Inhalation Solution S2 is indicated for the treatment and prevention of alterations respiratory tract, related to flu syndromes, cough, colds, allergic rhinitis and sinusitis; and allows symptoms more quickly. The combination of ingredients contained in **Inhalation Solution S2** assures a protective, emollient and hydrating action at the level of the respiratory epithelium and, in association with the slight hypertonicity of the solution, promotes better removal of excess mucus and contributes to the normalisation of mucociliary clearance, thus reducing contact between the mucosa and pathogenic and irritating agents, hence the risk of allergic irritations, bacterial and viral infections. Furthermore, the antioxidant action of plant extract of Perilla promotes restoration the physiological balance of the upper respiratory tract.

Nebulisation of the **Inhalation Solution S2** is excellent for cleaning and hydrating the airways and to promote elimination of excess mucus. **Inhalation Solution S2** is preservative free.

DIRECTIONS FOR USE:

1. Ready for use solution, do not dilute
2. Blow your nose before starting the session
3. Bend the single-dose ampoule both ways and detach it from the strip it is attached to
4. Slightly shake the single-dose ampoule before use
5. Turn and break off the cap of the ampoule
6. Apply slight pressure on the sides of the single-dose ampoule to squeeze the solution into the nebuliser cup (aerosol device)
7. During the session breathe regularly, duration is 4-5 minutes approximately
8. Do not reuse any residue in the ampoule and/or nebuliser cup after use

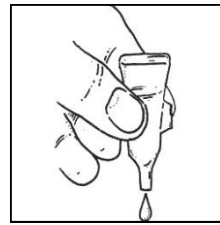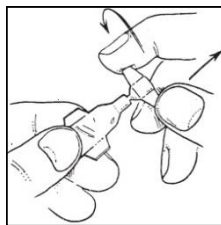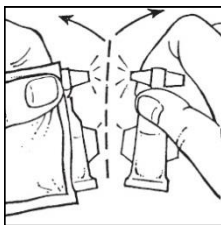

DOSAGE:

- Nebulise one 3 mL ampoule once/twice a day, or according to the doctor's opinion
- The product can be used with all types of aerosol devices (piston and ultrasonic) and nasal showers

COMPOSITION:

Sodium chloride 2.0%, Hyaluronic Acid, Dry extract of Perilla (Perilla Frutescens) seeds titrated 2.5% in polyphenols, Distilled water of Boswellia (Boswellia Serrata), Distilled water of Grindelia (Grindelia Robusta), Distilled water of Elderberry (Sambucus Nigra), Sodium Dihydrogen Phosphate Monohydrate, Disodium Phosphate Dihydrate, Purified Water.

PRESENTATION: The package contains 10 single-dose 3 mL ampoule in 5-ampoule strips.

WARNINGS:

- ✓ The product is for inhalation use only, do not ingest or use as eye drops.
- ✓ Keep away from the reach of children.
- ✓ Consult with the doctor for use in paediatrics and for use in pregnancy and while breastfeeding.
- ✓ Do not use under 3 years.
- ✓ Clean your hands carefully before using the device.
- ✓ While squeezing the product into the nebuliser cup do not touch other surfaces with the dispensing end of the ampoule.
- ✓ Do not use the product in case of known incompatibility to even one single component of the formulation.
- ✓ In the event of disorders such as bronchospasm or cough, or any perception of irritation during use, discontinue treatment immediately and consult with a doctor or pharmacist.
- ✓ The administration of drugs by inhalation concomitantly with the inhalation solution should be carried out under the supervision of a doctor who can assess the possible interactions.
- ✓ Do not use the product after the expiry date indicated on the package.
- ✓ After opening the ampoule, immediately pour the content and do not reuse any residual product.
- ✓ Store unused ampoules in the box.
- ✓ In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established.

VALIDITY: 36 months with the package intact



Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italy



Rev.01 December 2025

Optional information:

