


INDICATIONS

IN-NOSE GREEN SPRAY from the IN-NOSE line is a medical device for nasal application consisting of a sterile hypertonic solution, preservative free, with hyaluronic acid and herbal dry extracts, in a spray bottle of 20ml.

IN-NOSE GREEN SPRAY is indicated in case of allergic rhinitis and cold syndromes for the cleaning of the nasal cavity, thanks to the hypertonic solution and the presence of the hyaluronic acid help the fluidification and removal of the mucus, restoring the mucociliary clearance. The combination of hyaluronic acid with the herbal dry extracts of Perilla and Polygonum performs a soothing and antioxidant action and protects the nasal mucosa from external irritant agents such as smoke, dust, dry air and pollens, possible causes of irritative and allergic disturbances.

INSTRUCTION FOR USE:

1. Carefully clean hands
2. Blow the nose before the use
3. Shake gently the spray bottle before the use
4. Remove the protective cap
5. Hold the spray bottle with the thumb under the base and the nozzle between two fingers
6. At first use, press the pump a few times, until a uniform spray is emitted
7. Tilt your head back slightly and insert the nozzle into a nostril
8. Spray and breathe gently with your nose at the same time
9. After use, clean and close the nozzle with its protective cap
10. The product is for single-patient use to avoid possible contamination

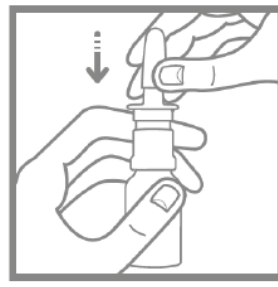

FREQUENCY OF USE: 1-2 nebulization per nostril 2/3 times a day, for 7-10 days or according to the doctor's opinion.

COMPOSITION: Sodium Hyaluronate, Sodium Chloride (2%), Perilla Frutescens dry extract (polyphenols $\geq 2,5\%$), Polygonum Cuspidatum dry extract (resveratrol $\geq 20\%$), Sodium Phosphate dibasic dihydrate, Sodium Phosphate monobasic monohydrate, purified water.

Subject:
Nasal Spray – SP1V, sterile solution in spray bottles of 20ml

Revision 04 – August 2025

Pages 2/2

PACKAGING: The box is composed of 20ml spray bottle, reclosable.

WARNINGS:

- The product is for inhalation use only, don't ingest or use otherwise than the indication of use.
- Once the bottle has been opened and used, the residual solution must be used within 3 months to avoid possible contamination of the product.
- Keep out of reach of children.
- Do not touch other surfaces, apart the nasal mucosa, with the dispensing end of the bottle.
- Do not use the product in case of known incompatibility to even one single component of the formulation.
- In case of adverse reactions (burning, irritation, etc.) during use, discontinue treatment and consult a doctor or pharmacist.
- Consult your doctor for pediatric use and for use during pregnancy and lactation.
- The administration of drugs or medical devices by inhalation concomitantly with the nasal spray should be carried out under the supervision of a doctor who can assess the possible interactions.
- Do not use the product after the expiration date indicated on the package.
- In the event of any serious claims or incidents concerning the device, it should be reported the incident to the manufacturer and the competent authority of the Member State in which the user is established, to provide more information on the safety of this medical device.

VALIDITY: 36 months with the package intact.




Enable Innovations S.r.l. - Via Claudio Tolomeo n°26; 41012 Carpi (MO), Italia



0425



STERILE A

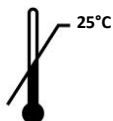

Rev.04 – August 2025

Optional information



:XXXXXXXXXX